


FITTING PRESCRIPTION LENSES TO B&S Swimming Goggles No.946101

Objective

To fit prescription lenses to the B&S Glazable Swimming Goggle No.946101

The Lens Bevel

The lens should be bevelled on a modern edger so that the bevel follows the flattest curve of the lens.

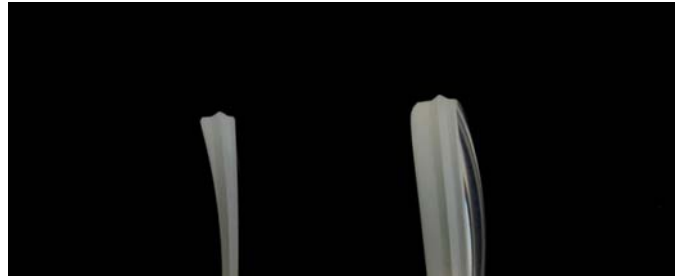

Instructions

The bevelled lenses must be fitted from the front and should follow the shape precisely. The lenses must be a tight fit.


(N.B. Practice on an old lens first to ensure shape and fitting accuracy.)



From the front, the silicon sealing ring/washer is placed around the inside rim against the rim-lips. *(N.B. If necessary, water can be used to hold it into position.)*

The Silicone seal ring fits inside the rim against edge Lip

The silicone seal/ring

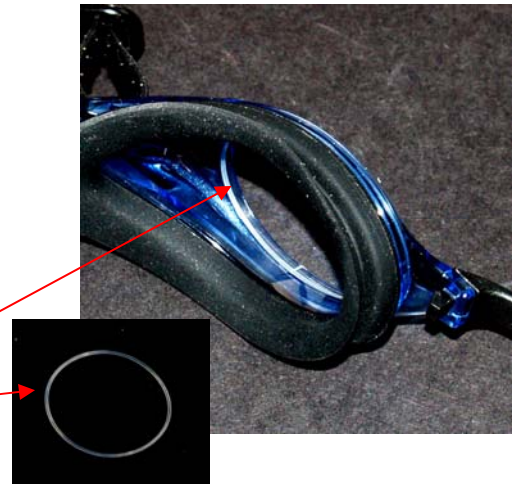

From the front press the lens into position so that the silicon ring/washer is squashed between the rim lip and the back of the lens.

